

**PerfectSeal**<sup>®</sup>  
Tissue Adhesive



## PerfectSeal<sup>®</sup> Tissue Adhesive

Add: A2-1 OF # 5 PLANT , NO.96 MUJIN ROAD, NINGBO  
HIGH-TECH ZONE, NINGBO CITY, ZHEJIANG PROVINCE.

Tel: +86-574-8822 5902

Fax: +86-574-88225901

E-mail: [sales@perfectseal.com](mailto:sales@perfectseal.com)

Http: [www.perfectseal.com](http://www.perfectseal.com)

**PerfectSeal**<sup>®</sup>  
Technology Protects Health

## Topical Skin Adhesive

**SECURE / STRONG / EFFECTIVE**

Fast & Strong medical adhesive  
for optimal wound closure



## COMPANY INTRODUCTION

ZHEJIANG PERFECTSEAL NEW MATERIAL TECHNOLOGY CO.,LTD. specialise in the designs, development , manufacture and distribution of medical adhesives and wound care products . It was originally set up to offer R&D, manufacturing and Sales & Marketing capabilities providing advanced medical adhesives and surgical dressings for healthcare companies. In strict accordance with the National Medical Device Management Regulations, a standardized purification plant has been built, and the quality is strictly controlled. In every aspect of management and production, the quality management standards of medical devices are strictly adhered to ensure the provision of quality products.

PerfectSeal is a registered brand in USA and Europe. It have passed ISO13485 and CE certificate. We are selling our patent technology products to Europe, America, Southeast of Asia, the Middle East, etc. PerfectSeal Products had highly praised from our clients. The company has been adhering to the core business philosophy of "Quality First, Sincere Service, Care For Health", continuously providing customers with the best quality products, creating value for society.

## OUR ADVANTAGES


PerfectSeal® Topical Skin Adhesive utilizes medical grade n-Butyl cyanoacrylate, an advanced monomer. It can polymerize into a thin protective film over wound edges when coming into contact with moisture in the skin. The polymerized cyanoacrylates create a strong,secure bond that holds the edges of a wound together to help facilitate the body's normal wound healing processes.

PerfectSeal® is a sterile, liquid skin glue that holds wound edges together. The film will usually remain in place for 5 to 10 days, and then naturally fall off your skin. PerfectSeal provide a safe and effective method of topical skin closure for surgical incisions.




Item No.	Description	Packing	Available Color
PAB-050	0.5ml n-Butyl Cyanoacrylate	1pcs/box,100boxes/ctn	Blue
PAB-100	1.0ml n-Butyl Cyanoacrylate	1pcs/box,100boxes/ctn	Blue
PAB-200	2.0ml n-Butyl Cyanoacrylate	1pcs/box,100boxes/ctn	Blue

## HOW TO USE

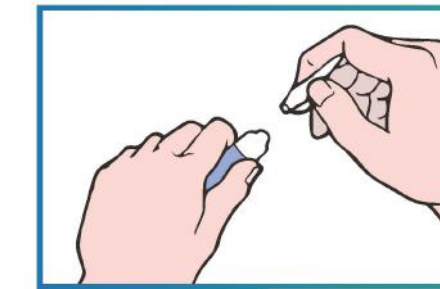


STEP1

Pat the wound dry with dry, sterile gauze to ensure direct contact of the adhesive to the skin.

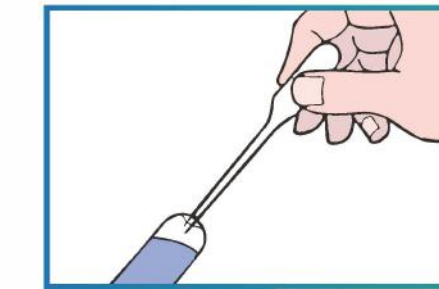


STEP2

Squeeze bottom of tube to saturate tip, bubble will form on top, then release pressure.

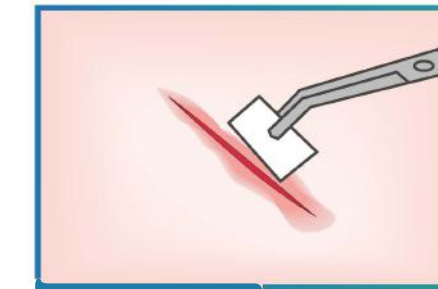


STEP3

Follow standard wound cleaning procedure using saline soaked lap sponge. Moisture aids in cure time.

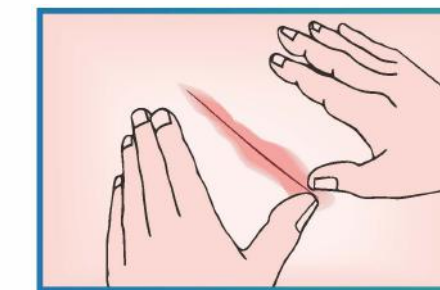


STEP4

Adapt wound edges

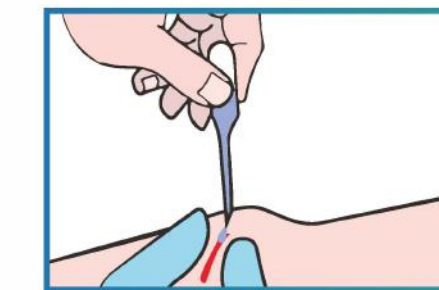

STEP5

Apply perfectseal © sparingly

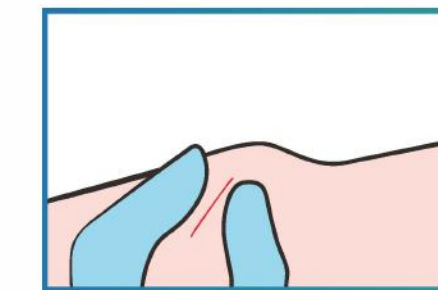


STEP6

Maintain manual approximation of the wound edges for approximately 15-60 seconds after the application.

## COMPARED TO LEADING COMPETITORS IN THE MARKET


Tensile Strength ( N )

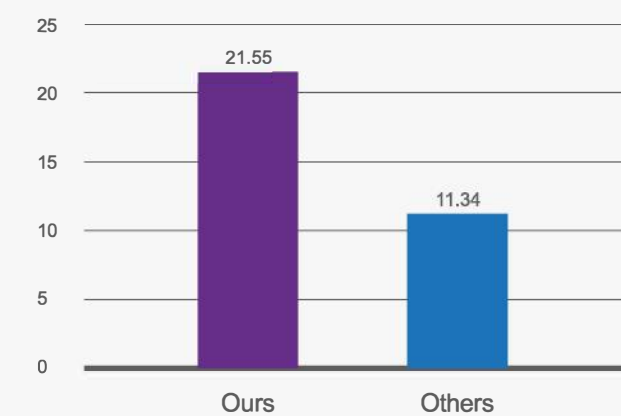

Faster Setting Time ( S )



## EVOLVING TO MAKE THINGS BETTER

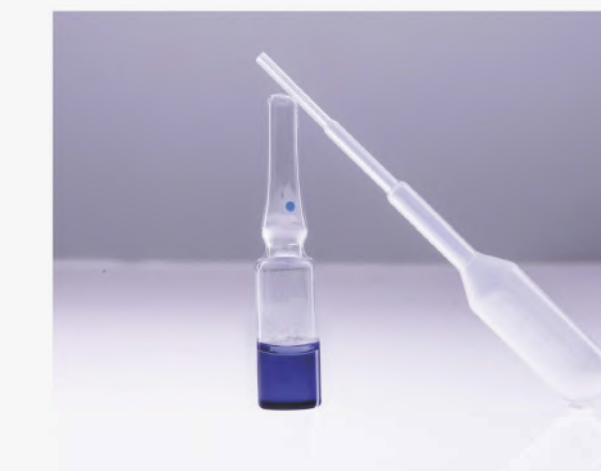

- Faster Setting Time. Offering less than 30 seconds dry time.
- Excellent tensile strength.
- Protects from infection.
- Precision control applicator.
- Better cosmetic outcome.
- Easy visual control with blue version
- The ideal tissue adhesive for closure of surgical incisions.